

**Localization** – Membrane

**Host Species** – Mouse

**Ig Class** – IgG / Kappa

### Intended Use

This antibody is designed for the specific localization of CD22/BL-CAM in formalin-fixed, paraffin-embedded (FFPE) tissue sections.

### Storage & Handling

Store RTU Vial at 2-8°C. Fresh dilutions for concentrated antibodies, if required, should be prepared prior to use and are stable for up to one day at room temperature (20-26°C).

### Working Principle

IHC is a two-step process wherein the primary antibody binds to the antigen of interest and that binding is detected by a chromogen. The primary antibody may be used in IHC using manual techniques or any Automated Staining System. Positive and negative controls should always be run simultaneously with all patient specimens.

### Product Description

CD22 (BL-CAM) is a type I transmembrane glycoprotein expressed on mature B cells and most B-cell lymphomas, including hairy cell leukemia and diffuse large B-cell lymphoma, but absent in classical Hodgkin lymphoma. It exists as two isoforms (130 and 140 kDa) produced via alternative splicing. Its extracellular region contains seven Ig-like domains that bind  $\alpha 2, 6$ -linked sialic acids on various cell types, though this binding may be masked by cis interactions on the same cell surface. CD22 expression is restricted to late-stage B-cell differentiation, making it a valuable marker for identifying mature B-cell malignancies. Intracellularly, CD22 contains six tyrosine residues within ITIM and activation-like motifs. Upon B-cell receptor (BCR) engagement, these sites are phosphorylated, allowing CD22 to modulate BCR signaling. It exerts inhibitory effects via SH2 domain-containing phosphatases (e.g., SHP-1) and promotes signaling through Src family kinases. Its regulatory function and lineage-specific expression make CD22 a key diagnostic and therapeutic target in B-cell biology.

### Material Supplied

CD22/BL-CAM antibody is affinity purified and diluted in PBS, pH 7.4, containing 1% BSA and 0.09% sodium azide.

### Material required But Not Supplied

- Xylene
- Isopropyl alcohol
- Positive charged slides
- Wash Buffer
- DI Water
- Antigen retrieval buffers
- Blocking Reagents
- Detection System
- Control Tissues
- Hematoxylin
- Mounting media
- Cover glass

### Working Reagent Procedure

- Ready-to-Use antibodies have been optimized for use with the recommended protocols and should not require further dilution.
- Concentrated antibodies must be diluted in accordance with the recommended protocol.

### Recommended Protocol

Refer the following table for the details on specific recommended protocol for this antibody.

<b>Control Tissue</b>	lymph node or spleen	<b>Antibody Incubation Time</b>	30-60 Minutes at RT
<b>Dilution factor</b>	<b>1:20-50</b> (Antibody Diluent: DH144)	<b>Retrieval Pre-treatment</b>	<b>Tris-EDTA based HIER</b> (AR9 Buffer: DH020)

### Precautions

*This product should be used by qualified and trained professional users only.*

Avoid microbial contamination of reagents to minimize non-specific staining. Never pipette reagents by mouth. Avoid contact of reagents and specimens with skin. If reagents or specimens come into contact with sensitive area, wash with sufficient amounts of water. Dispose of the unused reagents. This kit contain sodium azide at concentrations of less than 0.1%. Sodium azide is not classified as a hazardous chemical at these concentrations, but proper handling protocols should be observed. For more information on product hazards, precautions and waste disposal, *Material Safety Data Sheets* are available upon request.




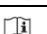
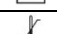
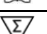



### Limitations

Improper tissue handling and processing prior to immunostaining can lead to inconsistent results. Variations in embedding and fixation or the nature of the tissue may lead to variations in results. Endogenous peroxidase activity or pseudo peroxidase activity in erythrocytes and tissue biotin may result in non-specific staining based on the detection system employed. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive with horseradish peroxidase systems. Improper counterstaining and mounting may compromise the interpretation of results. Interpretation of the staining result is solely the responsibility of the user. Experimental results should be confirmed by a medically-established diagnostic product or procedure. Evaluation must be performed by a qualified pathologist.

### Troubleshooting

For Technical Support contact us at +91 - 7506501122 or [info@dygnova.com](mailto:info@dygnova.com) or your local distributor to report unusual staining.

Doc No: DH/DS/CD439Rev.00

	Manufacturer Details		Use by Date		Lot/Batch Number
	Manufacturing Date		Consult Instructions for Use		Catalogue Number
	Temperature Limits		Sufficient for 'n' assays / tests		In-vitro Diagnostic Medical Device