

**Localization** – Nuclear  
**Host Species** – Mouse  
**Ig Class** – IgG2b, kappa

### Intended Use

This antibody is designed for the specific localization of PIT-1 in formalin-fixed, paraffin-embedded (FFPE) tissue sections.

### Storage & Handling

Store RTU Vial at 2-8°C. Fresh dilutions for concentrated antibodies, if required, should be prepared prior to use and are stable for up to one day at room temperature (20-26°C).

### Working Principle

Pituitary-specific transcription factor 1 (PIT1) is a lineage-defining transcriptional regulator essential for pituitary gland development and hormone gene expression in mammals. PIT1 belongs to the POU (Pit-Oct-Unc) family of transcription factors, a conserved group that governs key developmental processes; this family derives its name from the initial identification of PIT1 and OCT1 in mammals and Unc-86 in *Caenorhabditis elegans*. Structurally, PIT1 contains two highly conserved DNA-binding domains—the POU-specific domain and the POU-homeodomain—both of which are required for high-affinity, sequence-specific binding to regulatory elements of target genes, including those encoding growth hormone and prolactin. Functionally, PIT1 plays a central role in the transcriptional regulation of pituitary hormone genes, mediating prolactin and thyroid-stimulating hormone  $\beta$ -subunit gene expression in response to thyrotropin-releasing hormone and cyclic AMP-dependent signaling pathways.

### Material Supplied

PIT-1 -antibody is affinity purified and diluted in PBS, pH 7.4, containing 1% BSA and 0.09% sodium azide.

### Material required But Not Supplied

- Xylene
- Isopropyl alcohol
- Positive charged slides
- Wash Buffer
- DI Water
- Antigen retrieval buffers
- Blocking Reagents
- Detection System
- Control Tissues
- Hematoxylin
- Mounting media
- Cover glass

### Working Reagent Procedure

- Ready-to-Use antibodies have been optimized for use with the recommended protocols and should not require further dilution.
- Concentrated antibodies must be diluted in accordance with the recommended protocol.

### Recommended Protocol

Refer the following table for the details on specific recommended protocol for this antibody.

<b>Control Tissue</b>	Human pituitary or tonsil	<b>Antibody Incubation Time</b>	30-60 Minutes at RT
<b>Dilution factor</b>	<b>1:20-50</b> (Antibody Diluent: DH144)	<b>Retrieval Pre-treatment</b>	<b>Tris-EDTA based HIER</b> (AR9 Buffer: DH020)

### Precautions

*This product should be used by qualified and trained professional users only.*

Avoid microbial contamination of reagents to minimize non-specific staining. Never pipette reagents by mouth. Avoid contact of reagents and specimens with skin. If reagents or specimens come into contact with sensitive area, wash with sufficient amounts of water. Dispose of the unused reagents. This kit contain sodium azide at concentrations of less than 0.1%. Sodium azide is not classified as a hazardous chemical at these concentrations, but proper handling protocols should be observed. For more information on product hazards, precautions and waste disposal, *Material Safety Data Sheets* are available upon request.




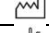


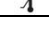
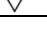
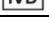
### Limitations

Improper tissue handling and processing prior to immunostaining can lead to inconsistent results. Variations in embedding and fixation or the nature of the tissue may lead to variations in results. Endogenous peroxidase activity or pseudo peroxidase activity in erythrocytes and tissue biotin may result in non-specific staining based on the detection system employed. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive with horseradish peroxidase systems. Improper counterstaining and mounting may compromise the interpretation of results. Interpretation of the staining result is solely the responsibility of the user. Experimental results should be confirmed by a medically-established diagnostic product or procedure. Evaluation must be performed by a qualified pathologist.

### Troubleshooting

For Technical Support contact us at +91 - 7506501122 or [info@dygnova.com](mailto:info@dygnova.com) or your local distributor to report unusual staining.

Doc No: DH/DS/IGG600Rev.00

	Manufacturer Details		Use by Date		Lot/Batch Number
	Manufacturing Date		Consult Instructions for Use		Catalogue Number
	Temperature Limits		Sufficient for 'n' assays / tests		In-vitro Diagnostic Medical Device